




Caring across generations since 1923

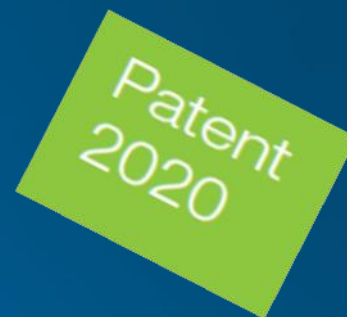

TOSAMA's Patented Reusable Applicator

PL: Vir, October 2021



Santé Health
Canada Canada






Reusable APPLICATOR: BENEFITS



- Patented reusable product.
- User-friendly:
Easy, safe and convenient to use.
- Enables the clean insertion of a digital tampon / applicator-free tampon into the vagina. Clean fingers 😊
- Suitable for the insertion of all sizes of digital / applicator-free tampons (Mini, Normal, Super)
- Suitable for organic cotton, cotton and viscose applicator-free tampons.

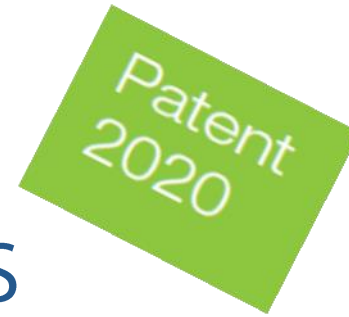

Reusable APPLICATOR: BENEFITS

Patent
2020



- Because we care about our planet.
- Less plastic in the environment (mountains, seas, lakes, oceans, ...)





Reusable APPLICATOR: CLAIMS

MEDICAL-GRADE / FOOD-GRADE MATERIAL

BPA-FREE



LATEX-FREE



PHTHALATE-FREE



MULTIPLE USE (at least 200 uses), LESS PLASTIC



LESS PLASTIC

Reusable APPLICATOR: MATERIAL COMPONENTS

MEDICAL-GRADE / FOOD-GRADE MATERIAL for our patented "Reusable Applicator"

Reusable Applicator COMPONENTS: 3 Parts

„Lipstick“ ergonomic cap with a matt finish.

A pusher with a smooth surface that is easy to clean.

hinge

A holder that holds the tampon in the correct position.



Reusable APPLICATOR: HOW TO USE

1. Wash your hands well.



2. Gently pull the pusher out of the top.
3. Open the hinge.



4. Unwrap your tampon of choice and place it into the holder.



Reusable APPLICATOR: HOW TO USE

5. Ensure that the tampon string is properly positioned – coming out of the string hole.



6. Now relax, adopt a comfortable position, remove the lipstick cap, insert the holder into your vagina and press the pusher.



7. Remove the holder and the pusher from your body, wash them with warm water and gentle soap, then ensure you dry everything thoroughly.
8. Put the lipstick cap back on and store the applicator in your toilet bag for the next use.





Reusable APPLICATOR: after your period, prepare it for the next cycle

1. Take the lipstick cap off and wash it with warm water and gentle soap. We recommend you disinfect it with an alcohol-based disinfectant (the same one you use for your hands) for a few seconds.



DON'T boil 

2. Wash the applicator with warm water and a gentle pH neutral soap. We recommend you pour boiled water over it for 1 minute.



Boil 

3. Dry thoroughly and put the lipstick cap back on.



Reusable APPLICATOR for A BETTER LIFE & PERIOD

We live in a country with beautiful nature and we want to keep it that way ... We offer it to you in our products

